

# D-Dry

Phonak drying and cleaning UV-C light kit



# Drying

## Function maintaining drying process with electrical heating

- Gentle process due to heat development and natural air circulation.
- Even heat development in the hygiene chamber permits the reliable escape of the collected moisture from the hearing systems and earmolds.
- Protect against moisture.
- Important. Regularly. And preferably daily.



## Time setting and automatic switch off for drying process

- Normal drying 3 hours for normal moisture collection.
- Intense drying 6 hours for severe moisture collection, e. g. at high humidity, after athletic activities, etc.
- The device switches off automatically after the completion of the drying process.
- Easy operation through the push of a button.



## Germ-reducing UV-C light, the „Hygiene PLUS“

- Irradiation of hearing systems and earmolds using UV-C light reduces the accumulation of germs and therefore the risk of skin infections in and around the ear.
- The effectiveness of the light source was confirmed by accredited test laboratories.
- Fully automatic 5 minutes UV-C cleaning process
- UV-C light safety shutdown when opening the cover.
- Long service life of the light source.

## D-Dry 3 in 1

- Compact sanitization chamber for drying, cleaning with UV-C light and storage.
- Drying with electrical heating.
- Time pre-selection for drying process – depending on the amount of moisture or the air humidity. 3 hours normal or 6 hours for more intensive care.
- Overheating protection.
- Cover with magnetic lock.
- Power supply: 5 V mains power supply.
- (100 to 240 V) or attached USB/mini USB cable from the computer.



## Power supply

- Mains operation and operation via the USB interface of your computer (cable is included in the scope of delivery).
- Available with EU, US, UK, Australian plug.